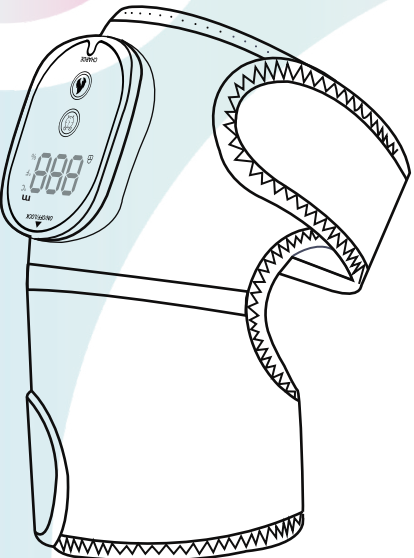
106mmx148mm

HEATED KNEE BRACE

USER MANUAL

MODEL: HLX102286/HLX10228601

Please read this manual carefully before use. After reading, please keep the manual properly for future reference.

Thank you for purchasing this product

Product Description

This heated knee brace is light, soft and breathable, comfortable to wear and safe to use. Our knee brace is ergonomically designed with extension straps for knees, elbows and shoulders. It can improve blood circulation, relax muscles, relieve physical fatigue, joint pain and discomfort.

NOTE:


Knee braces may relieve pain and discomfort, but are not therapeutic.

Product Parameters

Product Name	Heated Knee Brace
Working Voltage	5V
Charging Current	1500mA
Charging Time	2H
Battery Capacity	3000mAh

- 1 -

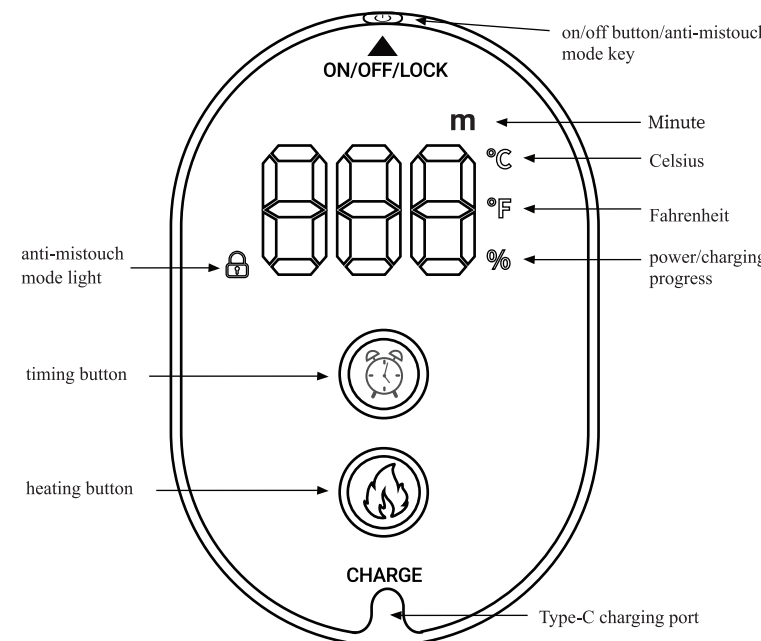BUTTON DESCRIPTION


NOTE:

The numbers on the controller screen indicate heating temperature (°C/°F), charging progress (%) and timing time (m).

- When the product is in use (non-charging state), the number indicates the heating temperature.
- During use, the anti-mistouch mode light is on, °C is on, the number represents the current power, and jumps back to % after a few seconds, and the number again represents the current heating temperature.
- When the timing function is used, the green light of m is on, and the number indicates the set time.
- When the product is in the charging state, the number represents the charging progress; when fully charged, the screen will display "FU".

- 2 -

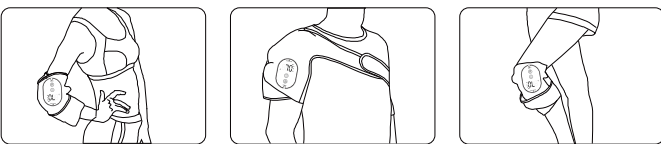APPLICABLE TO MULTIPLE PARTS

OPERATION GUIDE

1. Power On

Long press the power key to power on. If it cannot be turned on normally, it may be due to insufficient power, please try again after charging.

2. Wear

Secure the knee brace at the knee/elbow/shoulder position.

3. Heating Function

Click [°C] to turn on the heating function, and the system will automatically set it to the first gear, 45°C. There are six heating gears: 45°C<50°C<55°C<60°C<65°C<70°C. Click [°C] again to switch the heating gear, and the heating gear will be switched cyclically from low to high temperature.

NOTE: After the heating function is turned on, the temperature will rise slowly and will not directly jump to the temperature you set. When you set the temperature level, the temperature will stop rising or falling after rising to the level you set.

4. Timing Function

Click [m] to start the first gear of the timing function: 15 minutes timing. The timing function has three gears: 15min<30min<45min. Click [m] again to switch the timing of 30min and 45min in turn. Click [m] fourth time, the screen will display 0min, which means that the timing function is turned off. If you want to turn on the timing function, click [m] again.

- 3 -

5. Anti-mistouch Touch Mode

After setting the temperature level and timing level, short press the switch button, [m] will light up, and the anti-mistouch function will be turned on. At the same time, °C lights up, the number represents the current power, and jumps back to % after a few seconds, and the number again represents the current heating temperature. You cannot operate the controller at this time to prevent accidental button touches during product use.

If you want to operate the controller, please short press the power button to turn off the anti-mistouch mode. At this time, [m] turn off.

6. Switch Between °C and °F

Long press [°C] and [°F] for two seconds at the same time, the temperature unit will switch from °C to °F.

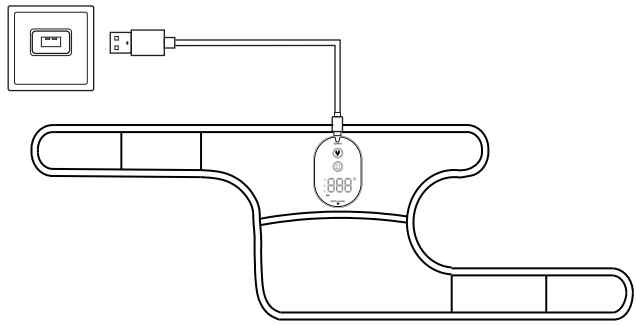
7. Turn Off

After use, long press the power button to turn off the knee brace. (If you set the timing function, the machine will automatically shut down when the time is over, without your manual operation.)

8. Storage

Keep the knee brace, extension straps and charging cable securely together so you don't lose accessories.

ABOUT CHARGING

- 4 -

- When the screen shows that the power is lower than 20%, please charge it in time.
- Please make sure the knee brace is turned off before charging.
- Take out the equipped charging cable, insert it into the Type-C interface of the controller, and connect the USB interface of the charging cable to the charging device.
- The display screen of the controller lights up, indicating that the knee brace has started charging, and the number represents the current power. During the charging process, the number will slowly increase, indicating the charging progress.
- The display screen of the controller displays "FU", indicating that the knee brace is fully charged.

NOTE:

Do not operate the controller during charging. Our knee brace cannot be used while charging.

PRECAUTIONS

- Wear thin clothes when using, such as shirts or yoga clothes; do not wear thick clothes, too thick clothes may affect the effect.
- Do not wear this product directly on your body (without the barrier of clothes), it will burn your skin.
- It is not recommended to use the high temperature gear for a long time (may cause burns or skin damage).
- Do not expose the knee brace to sunlight or fire, so as not to damage the thermal conductor of the insulation.
- Do not disassemble or modify the heating foil and control circuit, so as not to cause artificial damage to the heating element.

SAFETY PRECAUTIONS

- When starting to use this product, it is recommended to select the low temperature mode. Slight skin redness and fever after use are normal. However, if severe skin redness or swelling occurs after use, discontinue use immediately and consult a doctor.
- It is recommended to avoid direct contact with the skin. When the product is used for a long time, its temperature is very high. Be careful of burns.
- It is recommended to start from the lowest temperature and gradually adjust to an appropriate temperature according to individual conditions to avoid skin burns for people who are not sensitive to heat.
- If the user has any discomfort during use, such as dizziness, headache, asthma, etc., please stop using it immediately and analyze the cause of the discomfort.
- When not in use, please turn off the knee brace and store it properly.
- This product is a household appliance and cannot replace medical equipment or medicine.
- Please pay attention to operating in accordance with the instructions, and regular operation may lead to safety accidents or property damage.

- 5 -

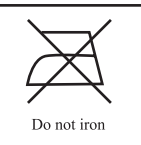
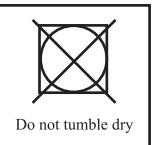
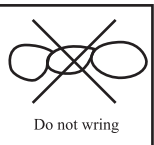
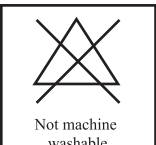
⚠ WARNING

CONSULT A DOCTOR BEFORE USE:

- Patients suffering from osteoporosis, fractures, sprains, acute pain and other leg diseases;
- Patients with nerve disorders in the legs or sensory disturbances in the skin;
- Pregnant or postpartum women;
- Heart disease patients;
- The product has a heating function, people who are not sensitive to temperature must pay attention.

ABOUT CLEANING

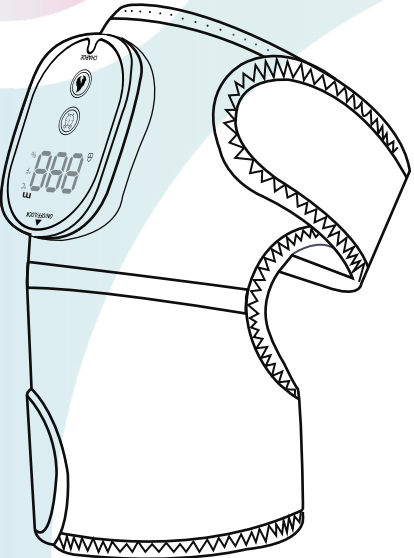
- Before cleaning, please make sure to turn off the power completely.
- When cleaning, wipe the knee brace with a damp cloth, but be careful not to wet the controller part. Do not use strong cleaners or hard brushes.
- Lay flat to dry.
- This product does not support machine wash or hand wash.
- Do not dry clean to avoid damage to the heating element.

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